 **AMSCO**

**hydrogen peroxide
sterilant**

ACTIVE INGREDIENT
Hydrogen Peroxide 30%
INERT INGREDIENTS 70%

**KEEP OUT OF REACH OF CHILDREN
DANGER**

STATEMENT OF PRACTICAL TREATMENT: IN CASE OF CONTACT IMMEDIATELY FLUSH AFFECTED AREA WITH PLENTY OF WATER FOR AT LEAST 15 MINUTES. CALL A PHYSICIAN. REMOVE AND WASH CONTAMINATED CLOTHING AND SHOES PROMPTLY AND THOROUGHLY. IF SWALLOWED DRINK PLENTY OF WATER. HARMFUL OR FATAL IF SWALLOWED. DO NOT GET IN EYES.

SEE SIDE PANEL FOR ADDITIONAL PRECAUTIONARY STATEMENTS

8.45 FL. OZ. (250 mL) NOT FOR SALE OR USE AFTER:

PRECAUTIONARY STATEMENTS

HAZARDS TO HUMANS AND DOMESTIC ANIMALS: CORROSIVE TO EYES, SEVERE IRRITATION TO EYES, SKIN, NASAL PASSAGES, THROAT, LUNGS, AND INTERNAL ORGANS. WEAR RUBBER OR VINYL GLOVES AND SAFETY GOGGLES OR GLASSES WHEN HANDLING.

PHYSICAL AND CHEMICAL HAZARDS: STRONG OXIDIZER. AVOID CONTACT WITH COMBUSTIBLE MATERIALS. DRYING OF THIS PRODUCT ON CLOTHING OR COMBUSTIBLE MATERIALS MAY CAUSE FIRE. AVOID CONTAMINATION FROM ANY SOURCE INCLUDING METALS, DUST AND ORGANIC MATERIALS. SUCH CONTAMINATION MAY CAUSE RAPID DECOMPOSITION, GENERATION OF LARGE QUANTITIES OF OXYGEN GAS, AND HIGH PRESSURES. IN CASE OF FIRE USE WATER ONLY. IN CASE OF SPILL OR LEAK FLUSH AWAY BY FLOODING WITH WATER APPLIED QUICKLY TO ENTIRE SPILL OR LEAK.

DIRECTIONS FOR USE

GENERAL CLASSIFICATION. IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

THIS PRODUCT IS INTENDED FOR STERILIZATION OF INSTRUMENTS CONTAINING NO LONG NARROW LUMENS, WHEN USED ONLY IN AMSCO HYDROGEN PEROXIDE STERILIZER AS DIRECTED IN THE EQUIPMENT MANUAL. ONLY TRAINED PERSONNEL MAY USE THIS PRODUCT AS DIRECTED.

STORAGE AND DISPOSAL

STORAGE: STORE IN ORIGINAL CONTAINER AT ROOM TEMPERATURE OR REFRIGERATED.

DISPOSAL: EMPTY BOTTLE SHOULD BE FLUSHED WITH WATER BEFORE DISCARDING.

MANUFACTURED BY AMERICAN STERILIZER COMPANY • ERIE, PA
EPA REGISTRATION NO. EPA ESTABLISHMENT NO.

AMSCO HYDROGEN PEROXIDE STERILANT CARTRIDGE LABEL

ACCEPTED
with COMMENTS
in EPA Letter Dated:

JAN 11 1990

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide
registered under EPA Reg. No.
5749-7

BEST AVAILABLE COPY

**STERILIZER
EQUIPMENT MANUAL**

ACCEPTED
with COMMENTS
in EPA Letter Dated:

JAN 1 1967

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide
registered under EPA Reg. No.

5749-7

8031 10-7 117

CONTENTS

1.0	CYCLE DESCRIPTION	1
1.1	General	1
1.2	Step by step	1
1.3	Safety	1
2.0	INSTALLATION	3
2.1	Summary of Warnings and Cautions	3
2.2	Utility and Space Requirements	3
2.3	Unpacking	3
2.4	Pre-use Checkout	4
3.0	OPERATING INSTRUCTIONS	5
3.1	Before Operating This Equipment	5
3.2	AMSCO Hydrogen Peroxide Sterilant	5
3.2.1	Precautionary Statements	5
3.2.2	First Aid	5
3.2.3	Directions for Use	6
3.3	Preparation of the Sterilizer	6
3.4	Preparation of the Load	7
3.4.1	Sterilizable Items	7
3.4.2	Wrapping Materials	7
3.4.3	Cleaning	7
3.4.4	Packages	7
3.5	Loading the Sterilizer	7
3.6	Sterilization	8
3.7	Control Monitoring	9
3.7.1	Cycle Status Lights	9
3.7.2	Alarm Lights	9
3.8	Determining Efficiency of Sterilization Process	11
4.0	PREVENTATIVE MAINTENANCE	12
4.1	Daily	12
4.2	Weekly	13
4.3	Quarterly	13
5.0	TROUBLESHOOTING	14
6.0	REPLACEMENT PARTS	15
6.1	Spare Parts List	15
6.2	Installing A Chamber Door Gasket	15
7.0	Figures	16

ACCEPTED
with COMMENTS
in EPA Letter Dated:

11 1968
Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide
registered under EPA Reg. No.
5749-7

LIST OF ILLUSTRATIONS

- Figure 1. AMSCO Hydrogen Peroxide Cycle
- Figure 2. Sterilizer front and left side.
- Figure 3. Sterilizer Lower Cabinet
- Figure 4. Sterilizer, rear view
- Figure 5. Vacuum pump
- Figure 6. Proper Oil Level
- Figure 7. Sterilant Reservoir Bottle in Place
- Figure 8. Suggested Wrapping Procedure for Instrument Trays
- Figure 9. Manual Vacuum Break Valve
- Figure 10. Controls, Status Lights, and Alarm Lights
- Figure 11. Bacteria Filter and Prefilter
- Figure 12. Door Gasket

ACCEPTED
with COMMENTS
in EPA Letter Dated:

JAN 11 1990

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide
registered under EPA Reg. No.

5749-7

2710-7

1.0 CYCLE DESCRIPTION

1.1 GENERAL

The AMSCO Hydrogen Peroxide Sterilizer is a low temperature, vapor-phase sterilizer for the rapid sterilization of wrapped metal surgical instruments, (such as delicate microsurgical instruments) containing no long narrow lumens. The internal temperature is about 64°C. Cycle completion times are as short as 80 minutes. Extensive testing by AMSCO of the sterilization of these surgical instruments has proven the AMSCO hydrogen peroxide process to be a reliable, noncorrosive, nondulling sterilization method.

The sterilizer is designed to operate automatically, requiring the operator to only load, start, and unload the sterilizer, and to perform a few routine maintenance functions. The sterilization cycle consists of injections of hydrogen peroxide vapors into the evacuated chamber, followed by dwell periods to allow the vapors to penetrate wraps, and to contact and kill any microorganisms. Cycle progress is communicated to the operator by lights indicating ready to begin, currently progressing, and cycle complete. At frequent intervals during operation, internal checks are made of the status of the critical operating parameters: the temperatures of the chamber and the vaporizing unit, the integrity of the chamber vacuum, and the supply of sterilant. If any of these are not correct, a warning light is illuminated, and the cycle is aborted. The load cannot be considered sterile unless the problem is corrected and a new cycle is started, and runs to completion.

1.2 STEP BY STEP

The cycle steps are illustrated in Figure 1 as pressure excursions in time. The cycle starts at atmospheric pressure. The pressure drops as the chamber is evacuated. AMSCO Hydrogen Peroxide Sterilant is then injected, causing a small rise in pressure. The hydrogen peroxide vapor concentration in the sterilizer is approximately 15 mg/L. A dwell period (no change in pressure) allows hydrogen peroxide vapors to penetrate the load. The chamber is again evacuated. Repeated injection, dwell and evacuation sequences are executed. Bacteria-free air is then allowed to fill the chamber. Additional evacuations and air fills follow, flushing peroxide vapors from the system.

1.3 SAFETY

Because it is a vacuum cycle, there is no danger of chamber rupture from over pressurization or of sterilant vapors leaking to the room during the cycle. The door cannot be opened during the cycle, even if it is unlatched. At the completion of a normal cycle, the sterilant vapor concentration in the sterilizer is so low that it presents no hazard to the operator.

The sterilization efficacy of the sterilizer has been proven by standard AOAC sporicidal testing, wherein no surviving spores were found on 720 inoculated

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide
registered under EPA Reg. No.
5749-7

PM31

5749-7

6 of 27

carriers which were imbedded among surgical instruments in wrapped instrument trays and peel pouches.

5-112-7

2.0 INSTALLATION

2.1 SUMMARY OF WARNINGS AND CAUTIONS

The following is a summary of safety precautions which must be observed when operating or servicing this sterilizer. WARNINGS indicate the potential for danger to personnel, and CAUTIONS indicate the potential for damage to equipment. The precautions are repeated where applicable throughout the manual.

WARNING: THIS STERILIZER IS DESIGNED FOR USE WITH AMSCO HYDROGEN PEROXIDE STERILANT ONLY. USE OF ANY OTHER MIXTURE COULD CAUSE INEFFECTIVE STERILIZATION OR OTHER HAZARDS.

WARNING: AMSCO HYDROGEN PEROXIDE STERILANT IS A STRONG OXIDIZER. USE CARE IN HANDLING. FOLLOW THE PROCEDURES PROVIDED IN "OPERATING INSTRUCTIONS" ON PAGE 5.

CAUTION: Use only approved wrapping materials to wrap items to be sterilized. Other materials may reduce the sterilizing effectiveness of this process. (See "Wrapping Materials" on page 7)

CAUTION: Never use abrasive cleaning compounds, wire brush, or steel wool on sterilizer surfaces.

2.2 UTILITY AND SPACE REQUIREMENTS

The peroxide sterilizer is a self-contained, transportable device, which requires connection to only one 120 VAC, 60 Hz, 30 amp grounded outlet. Voltage as low as 110 VAC can be used, but momentary voltage excursions below 105 VAC will terminate the automatic operation of the sterilizer, requiring manual restarting.

Space around the sterilizer is needed to permit access to the sterilant reservoir (Figure 2), the vacuum pump (Figure 3), and the manual vacuum break valve (Figure 4) and to allow proper circulation of air to cool the vacuum pump and the controller electronics. Space for proper circulation is generally assured by providing space for human access.

2.3 UNPACKING

Upon receiving this equipment:

- Open chamber door and remove packing.
- Be sure chamber shelves are properly positioned on the rack.
- Open the left part of the upper cabinet and remove the packing.

2.4 PRE-USE CHECKOUT

1. Open the doors on the right side of the lower cabinet to access the pump (see Figure 3).
2. Check oil level in the vacuum pump. The oil should be just visible at the bottom of the sight glass (see Figure 5). If the level is low, add oil as instructed under "Quarterly" on page 13 of the "PREVENTATIVE MAINTENANCE" section. Use only Marvac Oil No. RO-1000.
3. Rotate the pump pulley by hand one or two turns to make sure the unit is not stuck. If it is impossible to rotate pulley by hand, contact an AMSCO service representative. Do not attempt to run the pump.
4. Check that the manual vacuum break valve (Figure 9) is closed.
5. Insert the plug of the power cord into a 120 volt, 60 Hz, grounded outlet, fused at 30 amp. Turn the power switch on (front panel, see Figure 2). The digital temperature display (front panel) will light up. If it does not, check that the power cord is properly connected, and that the correct power is actually available at the outlet. If the temperature display still does not come on, refer to the "TROUBLESHOOTING" on page 14, under "sterilizer will not operate". If no remedy is found, contact an AMSCO service representative.
6. Wait (about one hour) until the READY light (front panel) comes on. At this time the inside temperature of the sterilizer is about 64 °C.
7. Press the CYCLE START button. The IN PROGRESS light will light and the vacuum pump will start running.
8. Wait (about 10 to 30 minutes) until the NO STERILANT alarm light (front panel) comes on. This will occur after the the vacuum pump has successfully evacuated the sterilizer chamber. (The NO STERILANT light indicates that there is no sterilant in the sterilant reservoir; it is shipped empty.)
9. Stop the cycle and relieve the vacuum as instructed under VACUUM LEAK under "Alarm Lights" on page 7.
10. Turn the POWER switch OFF, and check the (now warm) oil level in the vacuum pump. If it shows an overfill condition (see Figure 6), drain enough oil to return the level to normal. (The draining procedure is given under "Quarterly" on page 13 of the "PREVENTATIVE MAINTENANCE" section.)

3.0 OPERATING INSTRUCTIONS

The information in this section, if followed carefully, will provide optimum equipment performance.

3.1 BEFORE OPERATING THIS EQUIPMENT

- Read this equipment manual and become thoroughly familiar with its contents.
- In-service training is required.

3.2 AMSCO HYDROGEN PEROXIDE STERILANT

3.2.1 PRECAUTIONARY STATEMENTS

DANGER

Hazards to Humans and Domestic Animals:

- Corrosive to eyes. Severe irritant to eyes, skin, nasal passages, throat, lungs, and internal organs.
- Wear rubber, or vinyl gloves and apron, and eye protection when handling.

Physical and Chemical Hazards:

- Strong oxidizer.
- Avoid contact with combustible materials. Drying of this product on clothing or combustible materials may cause fire.
- Avoid contamination of AMSCO Hydrogen Peroxide Sterilant from any source, including metals, dust and organic materials. Such contamination may cause rapid decomposition, generation of large quantities of oxygen gas, and high pressures.

Keep out of reach of children.

3.2.2 FIRST AID

- In case of contact with eyes, immediately flush eyes with plenty of water for at least 15 minutes. Call a physician.

Print

5749-1

10/27

- In case of contact with skin, flush skin with water. (Skin may temporarily turn white, and a stinging sensation may be felt. Skin will return to normal within 3 hours and stinging will dissipate, if contact time is short.)
- Remove and wash contaminated clothing and shoes promptly and thoroughly.
- If swallowed, drink plenty of water

In case of fire use water only.

In case of spill or leak, flush away by flooding with water applied quickly to the entire spill or leak.

3.2.3 DIRECTIONS FOR USE

- General Classification. It is a violation of Federal law to use this product in a manner inconsistent with its labeling.
- Use only to sterilize microorganisms on items described under "Sterilizable Items" in the "Preparation of the Load" section of this manual.
- The AMSCO Hydrogen Peroxide Sterilizer automatically dispenses the Sterilant to provide an average dose of 0.2 mL Sterilant per liter of sterilizer volume with each sterilization cycle. Attach the bottle of sterilant to the Sterilizer per instructions under "Daily" in the "Preventative Maintenance" section of this manual.
- Store in original containers at room temperature or refrigerated.
- Spent sterilant containers should be flushed with water before discarding.
- Unused AMSCO Hydrogen Peroxide Sterilant can be flushed down the drain with plenty of water.

3.3 PREPARATION OF THE STERILIZER

BEFORE CYCLE STARTING THE FIRST CYCLE OF THE DAY:

Perform daily maintenance, following instructions under "Daily" on page 12 in the "PREVENTATIVE MAINTENANCE" section. This includes:

1. Drain any water from the vacuum pump.
2. Check pump oil level.
3. Clean the inside surfaces and shelves of the sterilizer.
4. Check the level of AMSCO Hydrogen Peroxide Sterilant in the reservoir bottle, and replace with a fresh bottle if needed, as instructed under "Daily" in the "PREVENTATIVE MAINTENANCE" section.

P/M 31 3.4 PREPARATION OF THE LOAD 5749.7 11/27

3.4.1 STERILIZABLE ITEMS

The AMSCO Hydrogen Peroxide Sterilizer is designed for the rapid sterilization of wrapped surgical instruments, without dulling or corroding the cutting edges. Typical instruments which can be sterilized include scissors, knives, and burs. Specific examples are microsurgical scissors (iris, corneal, vitreous), cataract knives, tympanum perforators, and forceps. Items with long narrow lumens, such as endoscopes, suction cannula, and aspirating needles, should not be run in this sterilizer.

3.4.2 WRAPPING MATERIALS

CAUTION: Use only recommended materials to wrap items to be sterilized. Some absorbent materials (for example, muslin) may reduce the sterilizing effectiveness of this process.

Recommended materials include heat bonded polypropylene sheets (such as Kinguard®, Kimberly-Clark, Neenah, Wisconsin) and spunbonded olefin peel pouches (such as Tyvek®, E. I. duPont, Wilmington, Delaware).

3.4.3 CLEANING

The article to be sterilized must be decontaminated, cleaned, rinsed and dried before placement in the sterilizer.

3.4.4 PACKAGES

Items to be sterilized may be loosely grouped in trays and wrapped with recommended wraps. The tray should be perforated at sides, bottom, and top to allow vapors to pass into the tray and contact all surfaces of the load. Wrapping of the tray is required to prevent contamination of the load during storage following sterilization. A suggested wrapping procedure is shown in Figure 8. Items may also be packaged in peel pouches and the peel pouches stacked on edge in a basket.

3.5 LOADING THE STERILIZER

- Place packages on the shelves and arrange the load so that the sterilant vapor can circulate freely around each item.
- Leave an inch or more space between packages.

- Provide at least 2 inches of space between the chamber walls and the packages of the load.

3.6 STERILIZATION

Review the preceding sections of this manual before operating the sterilizer.

1. Plug the power cord into a grounded 120-V, 60 Hz, 30 amp outlet.
2. Turn on the power by pressing the POWER switch on the front panel.
3. Check that the manual vacuum break valve (Figure 9) is closed.
4. Allow about two hours for warm-up before loading or using the sterilizer. This time is required to heat the chamber to a uniform operating temperature. Close and latch the door during this warm-up period.
5. After the READY lamp has come on, place the load in the chamber and latch the door.
6. Press the CYCLE START switch. The sterilizer will automatically proceed through its cycle of evacuation, sterilant introduction, and sterilizing dwells. During this period the IN PROGRESS lamp is illuminated.
7. The COMPLETE lamp will light when the cycle is complete and the door may be opened. Open the door and remove the load.
8. A new load may now be placed in the sterilizer. When the door is closed, the controller automatically resets for the start of a new cycle. Proceed as instructed in steps 6 and 7.
9. If subsequent cycles are not going to be run now:
 - a. Press the POWER switch OFF.
 - b. Leave the door slightly open when not in use. This will extend the life of the gasket.

NOTE: The cycle may be interrupted at any time by pressing RESET. The controller will jump to the cycle termination step to flush out any sterilant vapors and prepare the sterilizer to be opened. Reinitiate the cycle by pressing CYCLE START after the READY light is on.

NOTE: If there is a power interruption during a cycle, the IN PROGRESS light will go out and will not go on again when power is restored. The cycle must be restarted, after power is restored, by:

1. Press the POWER switch off, then on again,
2. Wait until the READY light lights,
3. Press the CYCLE START switch.

5747-1
1527
The controller will return to the beginning of the cycle and run a complete cycle, without regard to the fraction of the cycle that was completed at the time of the power interruption.

3.7 CONTROL MONITORING

To ensure the validity of the sterilizing process, the automatic control continually monitors the cycle. The indicating lights on the front panel (see Figure 10) call attention to the "STATUS" of the sterilizer cycle (upper three lights) and to conditions that might prevent successful completion of a sterilization cycle (lower five "ALARM" lights). An explanation of each light follows, along with actions that should be taken, should an alarm condition occur.

3.7.1 CYCLE STATUS LIGHTS

- **READY** -- Chamber is at correct temperature; sterilizer can be loaded and started.
- **IN PROGRESS** -- Controller is operating; sterilizing cycle is progressing.
- **COMPLETE** -- Cycle is complete; sterilizer may be opened and unloaded. -

3.7.2 ALARM LIGHTS

1. VACUUM LEAK

Indicates air is leaking into the evacuated chamber.

OPERATOR SHOULD:

- Open the access door on the rear panel (Figure 4) and check that the manual vacuum break valve (Figure 9) is closed tightly.
IF it was NOT, break the vacuum and restart the cycle:
 - a. Slowly open the manual vacuum break valve. A loud hissing sound will be heard as air rushes through the valve into the evacuated sterilizer.
 - b. When the vacuum in the sterilizer has been relieved (hissing cannot be heard), close the manual vacuum break valve tightly.
 - c. Press the RESET switch, then, when the READY light comes on, press CYCLE START (front panel).

IF the manual vacuum break valve IS CLOSED tightly and the condition occurs, call an AMSCO service representative.

2. SHELL TEMPERATURE OUT OF RANGE

57197 10.1.27
Normally lit during the initial warm-up period, after the POWER switch is turned on, and before the READY light comes on. It is normally off at all other times.

IF this light comes on AFTER a cycle has begun,
OPERATOR SHOULD:

- a. Press the RESET switch. The controller will jump to the cycle termination step to flush out any sterilant vapors and prepare the sterilizer to be opened.
- b. Wait 30 minutes
- c. If the READY light comes on, press the CYCLE START switch, to restart the sterilization cycle.
- d. If the READY light does NOT come on, press the POWER switch off, and call an AMSCO Service representative.

3. VAPORIZER TEMPERATURE OUT OF RANGE

Indicates a malfunction in the vaporizer heater.
IF this light comes on during a cycle,

OPERATOR SHOULD:

- a. Press the RESET switch. The controller will jump to the cycle termination step to flush out any sterilant vapors and prepare the sterilizer to be opened.
- b. Wait 30 minutes
- c. If the READY light comes on, press the CYCLE START switch, to restart the sterilization cycle.
- d. If the READY light does NOT come on, press the POWER switch off, and call an AMSCO Service representative.

4. NO STERILANT

Indicates sterilant is not being delivered to the chamber.

OPERATOR SHOULD:

- Press the RESET switch.
- Check the sterilant reservoir bottle (Figure 7). If empty,
 - a. Replace with a new bottle of AMSCO Hydrogen Peroxide Sterilant per instructions under "Daily" in the "PREVENTATIVE MAINTENANCE" section.
 - b. Restart the cycle, when the READY light comes on, by pressing the CYCLE START switch.
- If the bottle is NOT empty, wait until the READY light comes on, then press the POWER switch off, and call an AMSCO Service Representative.

5. SLOW DRAW DOWN

Indicates that evacuation is taking much longer than normal. It could be an indication of a vacuum leak or a pump malfunction.

OPERATOR SHOULD:

- Open the access door on the rear panel (Figure 4) and check that the manual vacuum break valve (Figure 9) is closed tightly.
IF it was NOT, break the vacuum and restart the cycle:
 - a. Slowly open the manual vacuum break valve. A loud hissing sound will be heard as air rushes through the valve into the evacuated sterilizer.
 - b. When the vacuum in the sterilizer has been relieved (hissing cannot be heard), close the manual vacuum break valve tightly.
 - c. Press the RESET switch, then, when the READY light comes on, press CYCLE START (front panel).
- IF the manual vacuum break valve IS CLOSED tightly and the alarm condition occurs, break the vacuum as described above, then:
 - Inspect the Door Gasket; replace, if necessary (see "Installing A Chamber Door Gasket" on page 15).
 - Drain some oil from the pump and inspect it. If it is badly oxidized (dark brown or black), or contains water, drain and replace the oil with Marvac Oil No. RO-1000 (the oil changing procedure is given under "Quarterly" on page 13 in the "PREVENTATIVE MAINTENANCE" section).
 - If the oil did not need replacing, check the oil level and add oil as needed.
 - Restart the cycle by pressing RESET, then, when the READY light comes on, press the CYCLE START switch.
 - If the condition recurs, break the vacuum as directed above, and call an AMSCO service representative.

3.8 DETERMINING EFFICIENCY OF STERILIZATION PROCESS

The best procedure to determine the efficacy of a sterilizing process is the biological monitor.

The most reliable form of biological monitor for gas sterilization is one such as AMSCO's Spordi® and Spordex®, which employ dry spores of a hydrogen peroxide resistant species ... Bacillus subtilis (globigii) ... seeded in known populations and standardized so that a known resistance can be demonstrated. It is recommended that 100 µg of catalase be added per milliliter of culture medium to neutralize hydrogen peroxide residuals on the spore strips and allow more efficient recovery of any surviving spores. Spordi, Spordex, and recommendations for preparation of the recovery medium, are available through your local AMSCO representative ... ask for details.

WARNING: REPAIRS AND ADJUSTMENTS, OTHER THAN THOSE DESCRIBED IN THIS MANUAL, SHOULD BE ATTEMPTED ONLY BY TRAINED SERVICE TECHNICIANS FULLY ACQUAINTED WITH THIS EQUIPMENT. THE USE OF INEXPERIENCED, UNQUALIFIED PERSONS TO WORK ON THE EQUIPMENT OR THE INSTALLATION OF UNAUTHORIZED PARTS COULD CAUSE PERSONAL INJURY, RESULT IN COSTLY DAMAGE, OR REDUCE THE EFFICACY OF THE STERILIZATION PROCESS.

CAUTION: Allow chamber to cool to room temperature before performing maintenance in and around the chamber.

4.1 DAILY

Before turning power on (power switch OFF):

1. Drain any water from the vacuum pump at the oil drain plug (see Figure 5). Drain at least 50 mL from the pump. If water appears with the oil, continue to drain until only oil flows from the drain. As much as 100 mL of water may be in the oil chamber, if the unit was in continuous operation the previous day. The water will have settled to the proximity of the drain plug if the pump has been off for more than about 8 hours. **THE WATER MAY CONTAIN SOME AMSCO HYDROGEN PEROXIDE STERILANT. Avoid contact. DISCARD THE WATER DOWN A DRAIN, and FLUSH with at least 10 volumes of water.**
2. Check the oil level in the vacuum pump. Add oil as needed. Do not overfill. (See oil changing procedure under "Quarterly" on page 13.)
3. Wipe down the inside surfaces of the sterilizer and the shelves with a lint free cloth soaked in distilled or deionized water. Dry with a lint free cloth.

CAUTION: Never use abrasive cleaning compounds, wire brush, or steel wool on these surfaces.

4. Check the level of AMSCO Hydrogen Peroxide Sterilant in the reservoir bottle. Replace with a fresh bottle as needed. The sterilant bottle is located in a bracket inside the door on the left side of the cabinet (see Figure 2, and Figure 7).

WARNING: AMSCO HYDROGEN PEROXIDE STERILANT IS A STRONG OXIDIZER. USE CARE IN HANDLING. (SEE "AMSCO HYDROGEN PEROXIDE STERILANT" ON PAGE 5.)

- a. Open the access door and remove the (empty) bottle from its bracket.
- b. While holding the cap steady, turn the bottle clockwise (as viewed from the top) until it is free from the cap.
- c. Carefully separate the cap and feed tube assembly from the bottle, assuring that the feed tube is not under tension such that it would flip drops of sterilant around as it emerges from the bottle.
- d. Reverse the procedure to install the new bottle in the bracket. Exercise care in handling the feed tube to prevent kinking or stretching.

- e. Flush the used bottle with water before discarding.

4.2 WEEKLY

Watch the sterilizer while it goes through a complete sterilizing cycle. Be sure the indicating lights are working properly.

- **READY** -- Chamber is at correct temperature.
- **IN PROGRESS** -- Controller is operating.
- **COMPLETE** -- Cycle is complete.

If the cycle runs properly, but these lights do not light, replace defective light bulbs.

4.3 QUARTERLY

- **Oil the door hinge:** Place a few drops of heavy machine oil (SAE 20 or 30 motor oil) on the chamber door hinge pin. Work oil into the hinge by opening and closing the door several times.
- **Inspect the door gasket.** Replace it if it becomes deformed, brittle or cracked (see "Installing A Chamber Door Gasket" on page 15).
- **Change the pump oil.** Drain the oil from the vacuum pump and replace with Marvac Oil No. RO-1000¹, only. Refer to Figure 5.
 - To drain the oil, hold a receiving tray or bottle under the oil drain plug. Unscrew the plug and allow all the oil to drain. Replace the drain plug.
 - To refill or add oil, remove the exhaust filter housing, and pour oil into the exhaust opening, until the oil level is between the lines on the sight glass (see Figure 6).
- Replace the exhaust filter element of the vacuum pump with a standard toilet paper roll.
- Replace the bacteria-retentive air filter and prefilter located above the pump in the lower cabinet (see Figure 11).

¹ Marvac Scientific Manufacturing Co., 1045 Shary Circle, Concord, Calif. 95518.

P1131

5749-7

19-27

5.0 TROUBLESHOOTING

Lights on the sterilizer front panel provide indications of most trouble conditions. Please refer to "Control Monitoring" on page 9 for corrective actions to take when an alarm light is lit.

Some other conditions which might occur are listed below, along with suggestions for correcting the condition. If you are unable to correct the problem, or a problem occurs which is not described on the chart, please call your AMSCO representative. He will arrange to have your equipment promptly put into working order by a factory-trained representative. **NEVER PERMIT UNQUALIFIED PERSONS TO WORK ON THE STERILIZER.**

PROBLEM	SUGGESTED CORRECTION
Sterilizer will not operate	<p>Be sure Chamber Door is closed, latched and tightly sealed</p> <p>Be sure the power cord is properly connected at the power outlet.</p> <p>Check that the POWER switch is on (press upper end of switch).</p> <p>Check for power at the outlet; if necessary, correct the cause of power failure, and reset the circuit breaker.</p> <p>Check the voltage at the outlet; a 105 VAC minimum is required.</p> <p>Check the 25 amp fuse, located behind the microprocessor (see Figure 3).</p>
Controller does not operate	<p>Check the voltage at the outlet; a 105 VAC minimum is required. Even short excursions below 105 Volts will cause the controller to fail. When proper voltage is restored, restart the sterilizer.</p>
Neither the IN PROGRESS nor the COMPLETE light is on after a cycle was started	<p>A power interruption has occurred. If power has been restored, either the READY or the CHAMBER TEMPERATURE OUT OF RANGE light will be on. After the READY light comes on, press CYCLE START to reinitiate sterilization.</p>

p1131

5749-7

19-837

6.0 REPLACEMENT PARTS

6.1 SPARE PARTS LIST

<u>DESCRIPTION</u>	<u>PART NUMBER</u>
Oil for vacuum pump	(Marvac RO-1000 ²)
Fuse, 25 Amp	
Gasket, door	P-45047-091
Filter, bacteria	
Pre-filter	P-41285-091

6.2 INSTALLING A CHAMBER DOOR GASKET

Refer to Figure 12.

- Remove the old gasket and clean the groove. (The replacement gasket is cut to provide a tight fit.)
- Force the new gasket into the groove, a short section at a time; do not stretch it while doing so. Should the gasket appear too long, **DO NOT CUT IT**. Start over again, compressing short sections into the groove, until the entire length is inserted. **Never use sharp tools to push the gasket into the groove.**

² Marvac Scientific Manufacturing Co., 1045 Shary Circle, Concord, Calif. 95518.

PM31

5749-7
7.0 FIGURES

20427

)

)

)


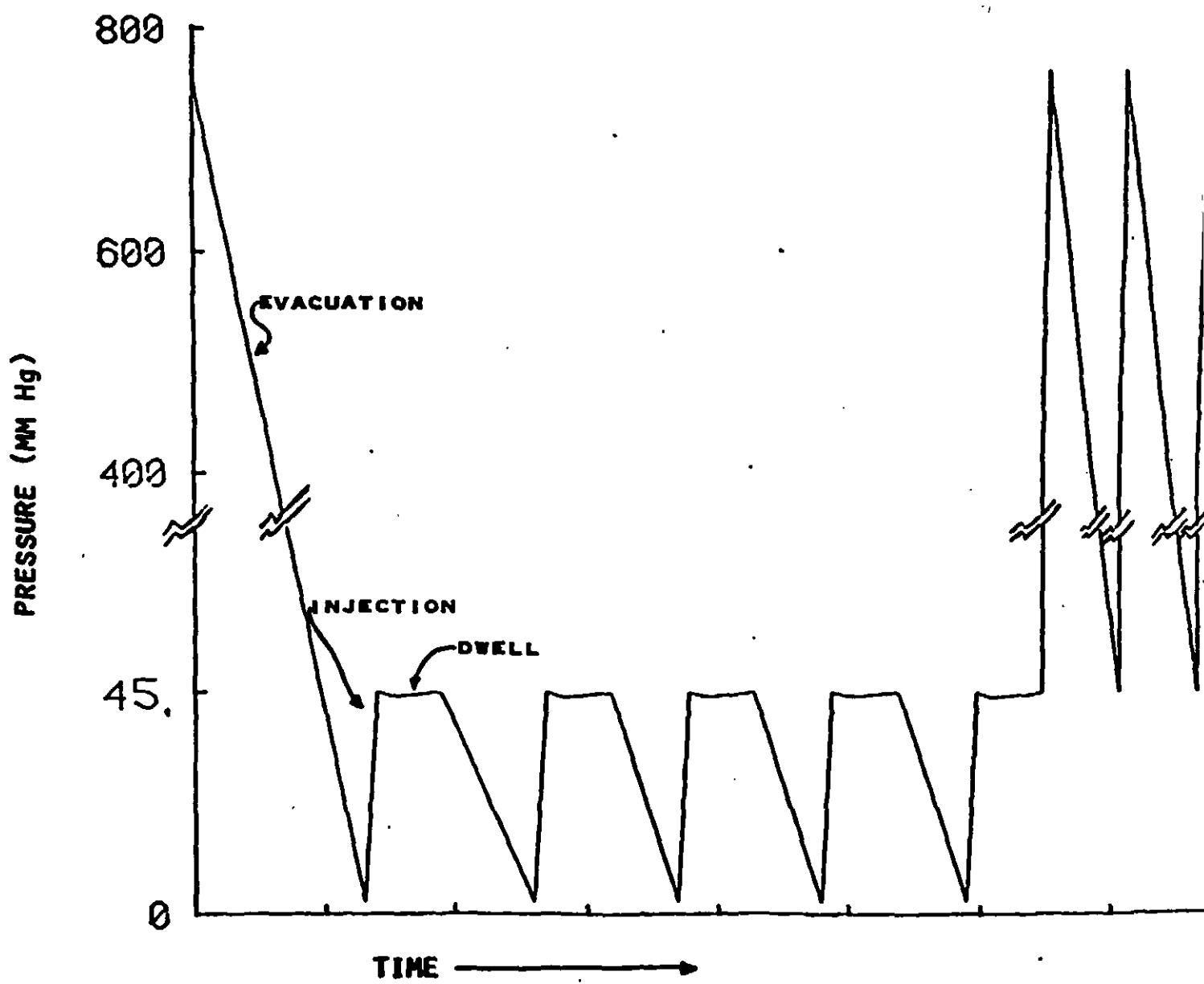


Figure 1. AMSCO Hydrogen Peroxide Cycle

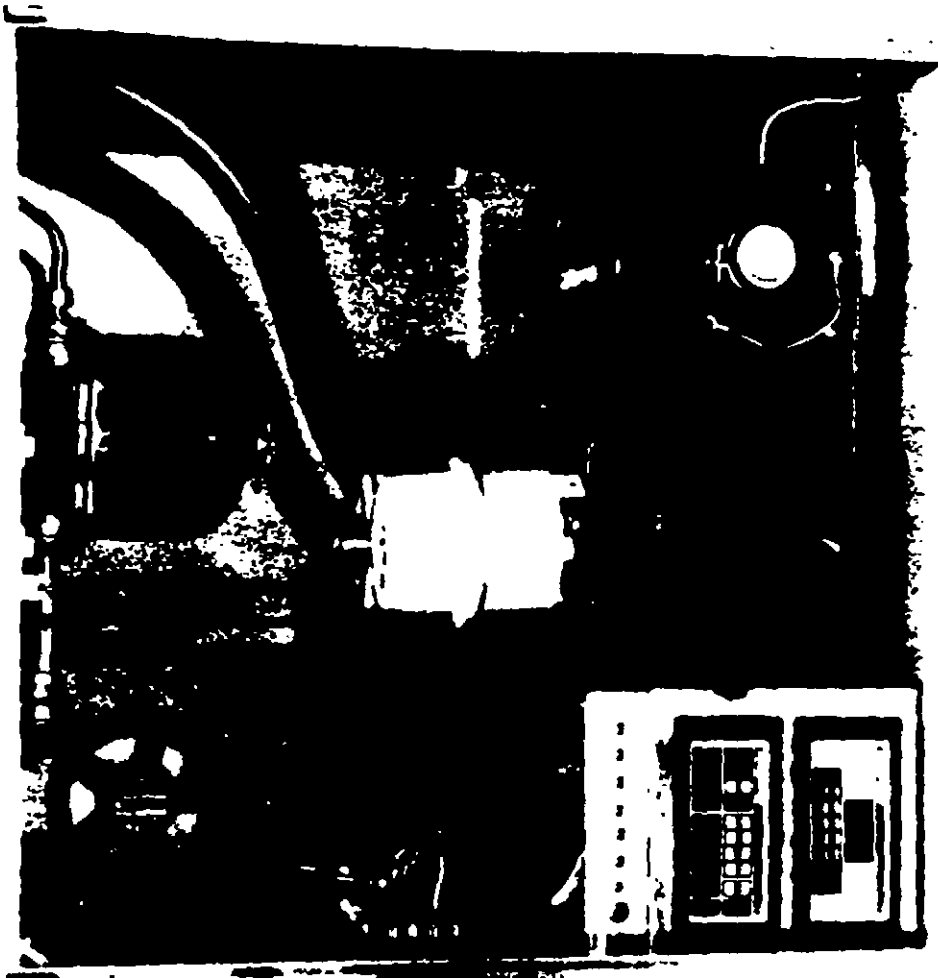


Right side view identifying in-cart features (lower cabinet)
Identify the controller, the vacuum pump and the three

Figure 3. Sterilizer Lower Cabinet

5749-7

23421

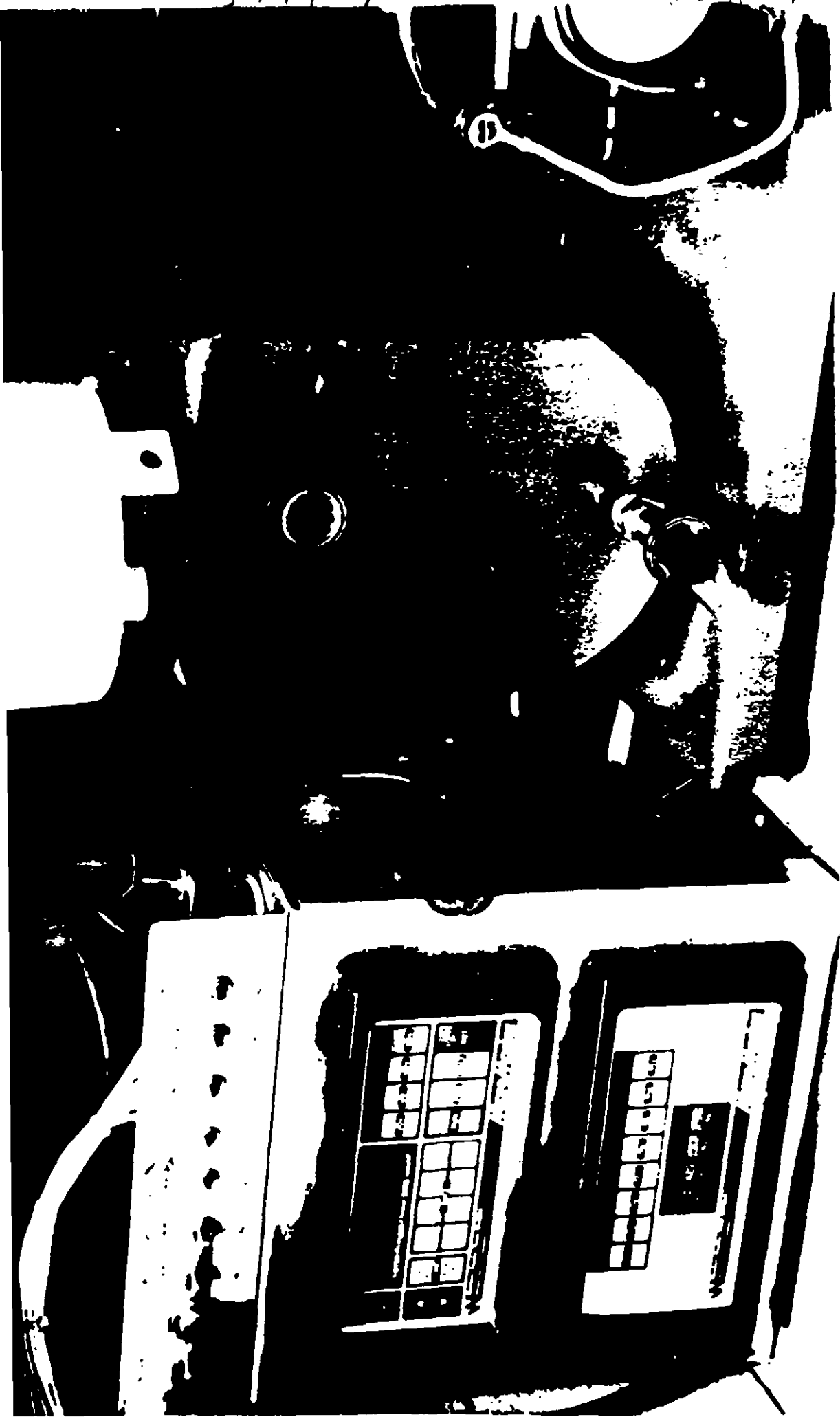

view of vacuum pump. Identify drain and fill points, exhaust and oil level sight glass.

5. Vacuum pump



Detail via
the bottle

Figure 7.



View of rear panel valve

Figure 1 - Manual Valve

P0131

5749-7




Front of sterilizer with door open. Identify the gasket.

Figure 12. Door Gasket

P1M31

5-199-1

27 of 27

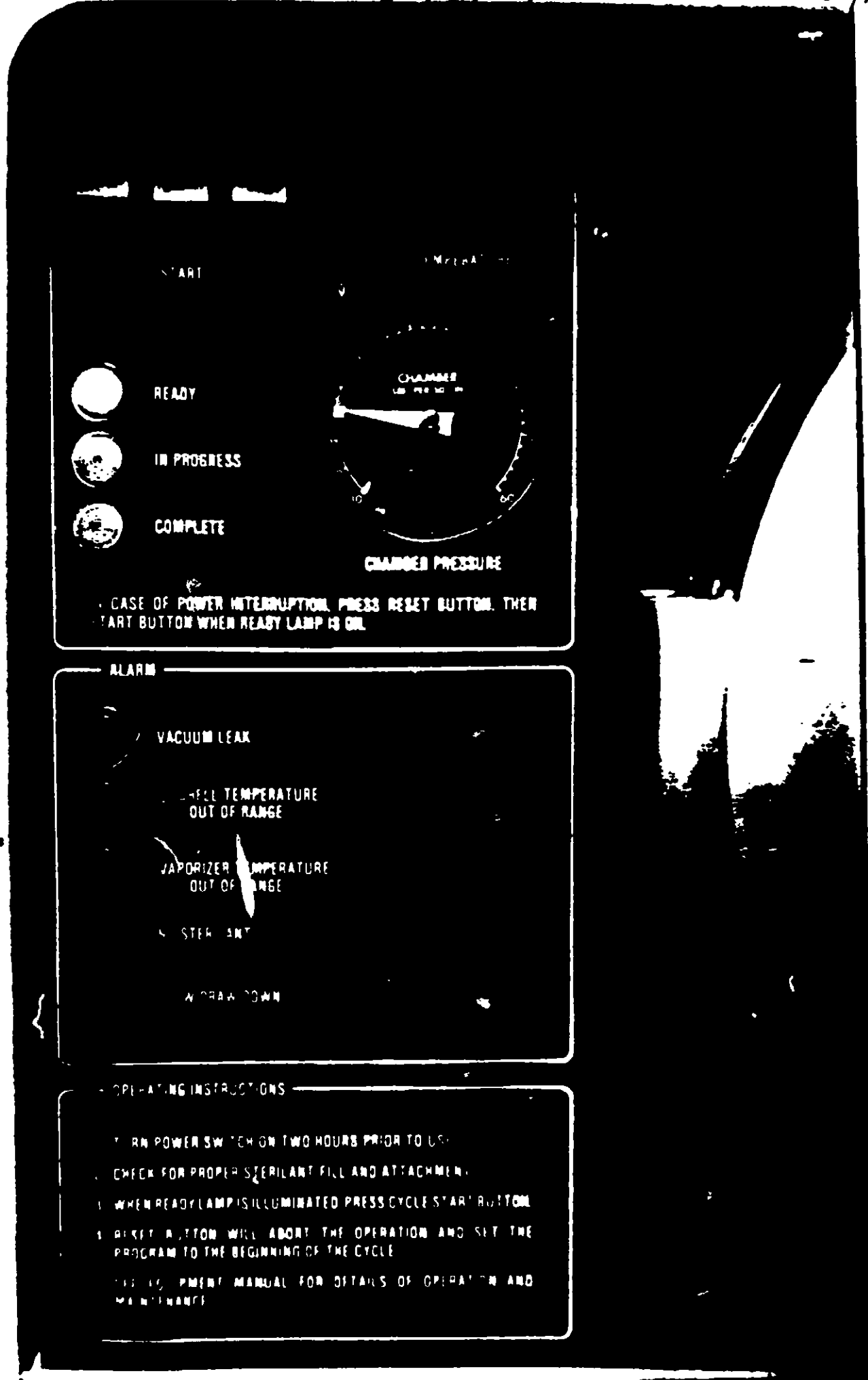

FRONT PANEL LABEL OF
AMSCO HYDROGEN PEROXIDE STERILIZER